
*Viotec*  
Product Manual

Viotec Health Care Design Inc  
[www.viotec.cc](http://www.viotec.cc)  
[Info@viotec.cc](mailto:Info@viotec.cc)

MADE IN CHINA

Product Name: IVE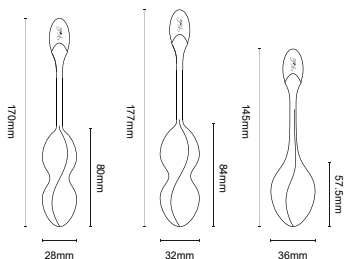

#### Specification

1. Material: Green Material nylon +50%GF and Silicone
2. Dimension:
  - \* single ball:  
 $\Phi 36 \times 145\text{mm}$ , weight: 53g
  - \* big double balls:  
 $\Phi 32 \times 177\text{mm}$ , weight: 76g
  - \* small double balls:  
 $\Phi 28 \times 170\text{mm}$ , weight: 88g
3. Color: Red, Pink
4. Net Product weight: 53g / 76g / 88g
5. Waterproof Grade: IPX7 Splashproof

#### How to use

1. Koro ball training is divided into three stages. It is composed of three balls with different weights and sizes. During use by stage, you should start with the easier one and then the more difficult one. In the training, you should use the single ball (weight: 53g) at the early stage, the big double balls (weight: 76g) at the medium stage and the small double balls (weight: 88g) at the later stage.

2. Before use, apply some water-soluble lubricant or other body lubricants on the product.

3. Push one end of the ball slowly into your vagina, and the outermost end of the ball should be at least 2cm away from the vaginal orifice.

4. Start training, stand and put your feet together, relax yourself so that the koro ball tends to slide under the action of gravity, contract the pelvic floor muscle to tighten the ball, control the force well and keep the ball in the vagina to prevent it from falling off. You can practice to repeatedly by combining the Kegel exercise to make your

vagina tight and elastic.

5. Apply the cleanser, rinse it with water, wipe its surface dry and store it.

#### Note

1. Clean and disinfect the product before and after the use. Rinse thoroughly with special antibacterial disinfectant or clean water, and dry with soft cloth or towels.
2. Do not use massage oil, hand cream, silicone-based lubricants. Only water-based lubricants are recommended. Do not use gasoline or acetone to wipe the product to avoid damage to the surface of the product.
3. Please keep the product in a clean and cool place away from direct sunlight, high-temperature and humid environments. Keep it out of the reach of the children.

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